


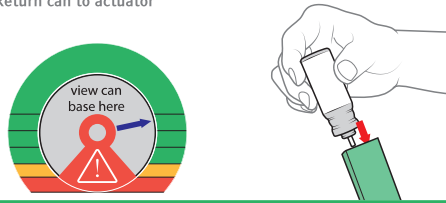
4


What's in the box




Arrow points to green zone  **SAFE TO USE**


- Inhaler has more than 50 doses left
- Return can to actuator




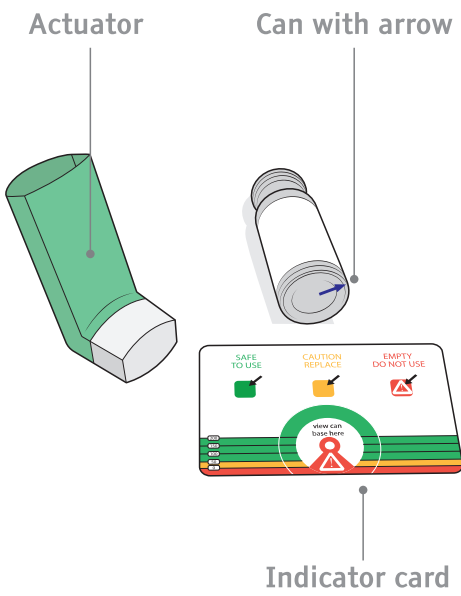
Arrow points to yellow zone  **CAUTION REPLACE**

- Inhaler has less than 50 doses left
- Get a new inhaler as soon as possible
- Return can to actuator

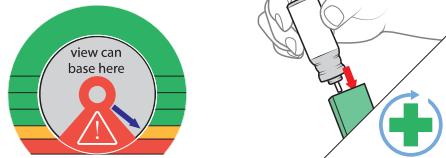

Arrow points to the red zone  **EMPTY DO NOT USE**

- Inhaler is empty
- Get a new inhaler immediately
- Discard the empty inhaler


A simple innovation that shows you how many doses are left in your inhaler...

 **PRESSPART**
www.presspart.com




...a revolutionary approach to patient safety

1

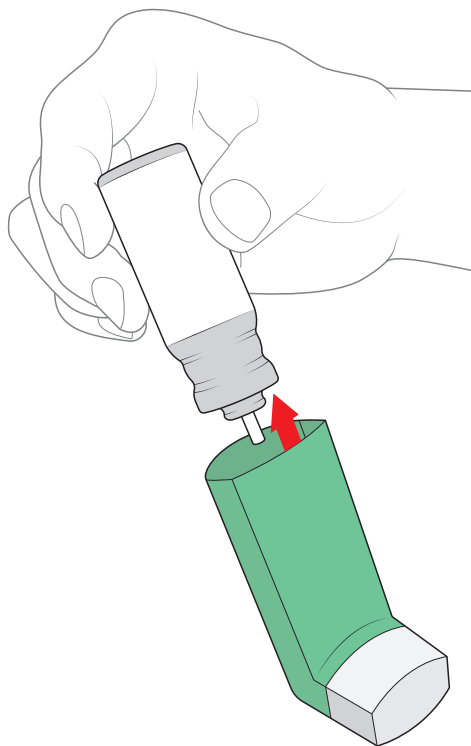



- Remove the can from the actuator

2

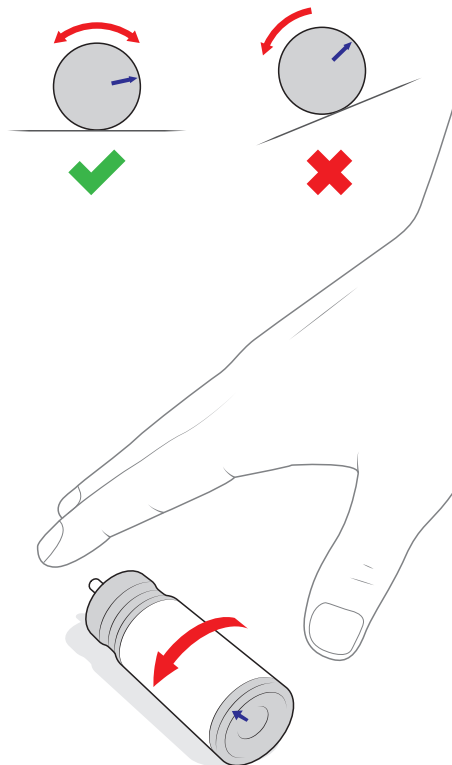

- Gently roll and release the can on a smooth and level surface
- Allow it to stop and settle naturally

3

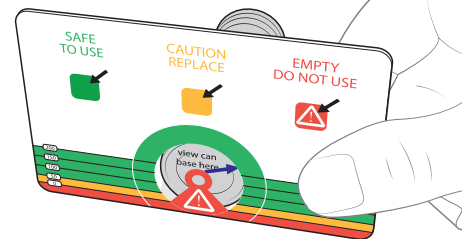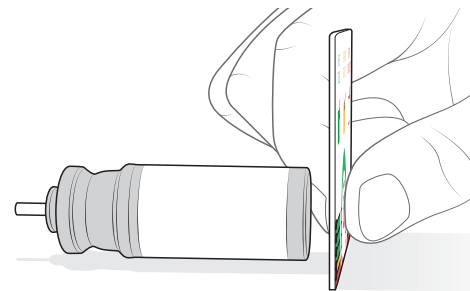

- To see how many doses are in your inhaler, align the transparent window on the indicator card with the end of the can
 - The arrow on the can points to a coloured band that shows how many doses are left
- * If you accidentally touch or move the can, repeat stage 2*