
AquaCare

Quick Start Guide

Dental Air Abrasion & Polishing Unit

Guide de démarrage rapide
Kurzanleitung
Guida di partenza rapida
Guía de inicio rápido
Инструкция по установке

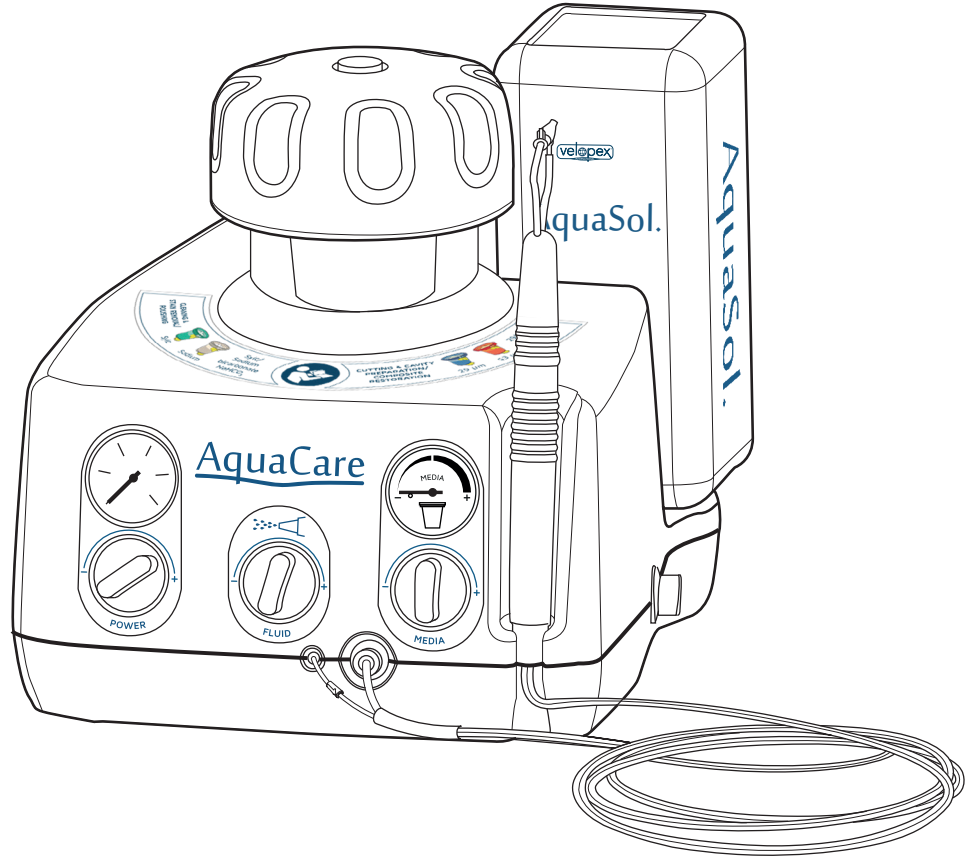AS90000

ATTENTION:

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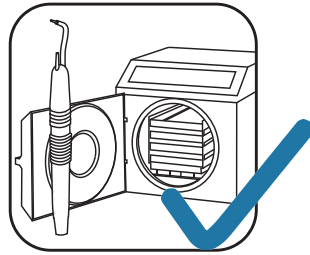
PROTECT SENSITIVE EQUIPMENT FROM DUST

Protégez les équipements sensibles de la poussière
\ Schützen empfindliche Geräte vor Staub
\ Proteggere le apparecchiature sensibili da polvere
\ Proteja el equipo sensible al polvo
\ Защитите оборудование от пыли

STERILISE HANDPIECE BETWEEN TREATMENTS


Stérilisez la pièce à main entre les traitements
\ Sterilisieren Sie das Handstück zwischen den Behandlungen
\ Sterilizzare il manipolo tra i trattamenti
\ Esterilice las piezas manuales entre tratamientos
\ Стерилизуйте наконечник перед каждым приемом

DON'T STEP ON TUBES

Ne marchez pas sur les conduites
\ Treten Sie nicht auf Leitungen
\ Non calpestare i tubi
\ No pise los tubos
\ Не наступайте на трубки

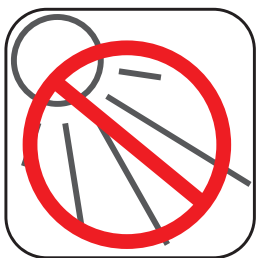DON'T PLACE IN SUNLIGHT

Ne l'exposez pas à la lumière directe du soleil
\ Vermeiden Sie direkte Sonneneinstrahlung
\ Non esporre direttamente alla luce del sole
\ No lo exponga al sol
\ Избегайте попадания прямых солнечных лучей

REPLACE TIP BETWEEN TREATMENTS

Remplacez l'embout entre les traitements
\ Tauschen Sie die Spitze zwischen den Behandlungen aus
\ Sostituire i puntali tra i trattamenti
\ Reemplace las puntas entre tratamientos
\ Меняйте пластиковые насадки перед каждым приемом

USE ONLY VELOPEX PARTS

Utilisez exclusivement des pièces Velopex
\ Verwenden Sie nur Velopex-Ersatzteile
\ Usare i componenti Velopex
\ Utilice únicamente piezas Velopex
\ Используйте детали Velopex

Uses

USES

(GB)

Preparation of pit and fissure for sealants. Removal of composites for restoration. Cavity preparation. Cleaning, polishing and stain removal. Etching.

ONLY QUALIFIED DENTAL PERSONNEL SHOULD USE THE AQUACARE. GIVE SPECIAL CONSIDERATION TO PATIENTS WITH:

- Respiratory difficulties
- Asthma
- Blocked nasal passages
- A cold
- Restricted sodium diet
- Any medical problem preventing breathing through the nose

Patients suffering from the above must consult their physician before treatment.

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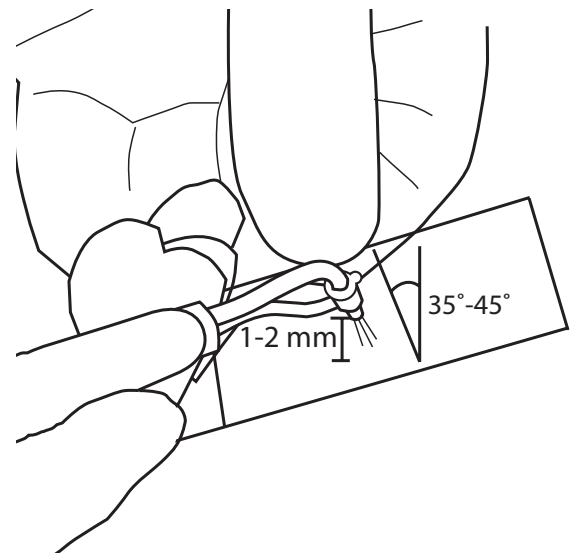
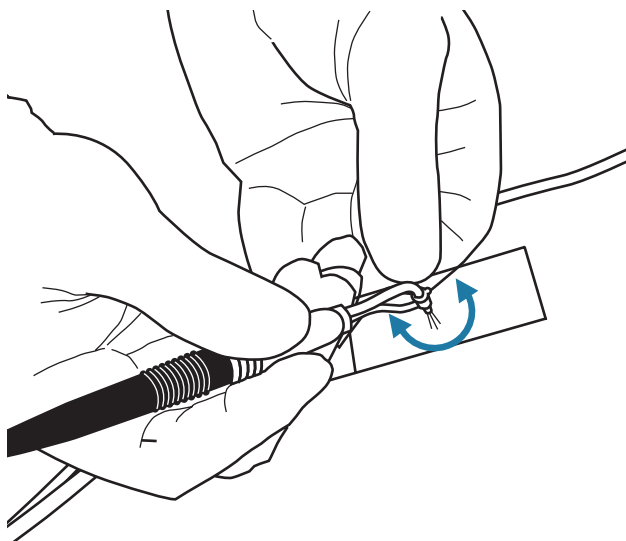
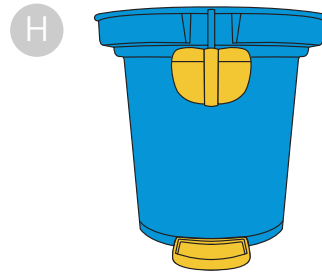
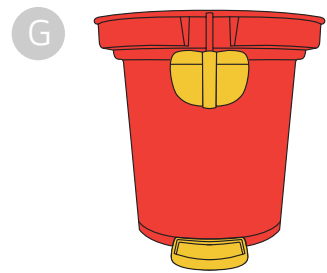
Suggested Treatment Methods

The following treatment suggestions have been prepared with the help of experienced practising dentists. The methods of treatment are advisory, and each dentist will quickly develop their own preferences in terms of powersetting, powder type, media volume, fluid volume, and the angle of the cutting nozzle.

Treatment Tips

The end of the cutting nozzle should be held approximately 1.5mm from the working surface. Avoid holding the nozzle at a right angle to the surface, as this will cause the powder to rebound and blur the view of the area. Angle the cutting tip towards the aspirator and instruct the Dental Assistant to follow the nozzle with the aspirator. The AquaCare operates with low noise, and has no vibration or heat at the cutting tip. Its ability to cut wet or dry gives it a high flexibility of use. Local anaesthetic can be offered to nervous patients, however it is generally not required as air abrasion is painless in most cases. The various treatments use a range of settings. The AquaCare has three dynamic controls which alter the cutting speed/cleaning rate. These are the power setting, the media volume, and the fluid volume. By increasing one setting as you decrease another, you can maintain the same cutting speed/cleaning rate but alter the treatment provided. Use less media and more power to improve viewing of the working area. Use more media and less power for sensitive areas.

Complimentary Cleaning Accessory

The AquaCare Dental Air Abrasion Cabinet offers a simple aid in support of your AquaCare abrasion unit to enable abrasion procedures to be done outside of the mouth. Preparation of dental attachments such as crowns, bridges or brackets can be cleaned and abraded before they are cemented in the mouth.

Preparations Before Treatment

WARNING: All people in the operating area must wear eye protection to BS EN 166 IF 415

WARNING: All people in the operating area except for the patient must wear respiratory masks to EN 149 FFP2S

1. It is recommended that a light coating of petroleum jelly is smeared around the patient's lips to prevent cracking and to retain some of the outflowing abrasive powder.
2. Rubber dam should be used to protect adjacent teeth and soft tissue.
3. Non-metal Matrix strips should be used to protect adjacent teeth during interproximal preparations.
4. A bib should be used to protect the patient's clothes from any out-flow of the material from the mouth during the course of a treatment.
5. Use a high-speed intra-oral aspirator, preferably with a flared tip.
6. An extra-oral evacuator can also be used with the extraction cone sited below the patient's chin.
7. IMPORTANT: To avoid unnecessary damage, when switching from Aluminium Oxide to Sodium Bicarbonate you must purge the powder.
8. It is recommended that AquaCare Cleaning/Cutting Fluid AquaSol is used with the AquaCare Fluid System. The 500ml bottle supplied with the machine should be discarded when empty and replaced. The fluid can be used with all recommended treatments using this instrument.

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Dental Air Abrasion & Polishing Unit

Recommended Accessories

WARNING: All replacement parts and consumables must be sourced from Velopex to ensure correct and safe functioning of the AquaCare.

Description	Part No
Equipment	
Breathing air filter unit	I/FIT 8200F
6mm Input Air Hose	I/ASS 8016F
Consumables	
ProSylc cleaning/polishing/desensitising bio-active powder, Pack of 4	I/PDR 0034F
ProClean cleaning/polishing sodium bicarbonate, Pack of 4	I/PDR 8014F
ProCut Fine cutting powder 29 micron aluminium oxide, Pack of 4	I/PDR 8025F
ProCut+ Coarse cutting powder 53 micron aluminium oxide, Pack of 4	I/PDR 8024F
Glass Slides (for cutting practice), Pack of 5	I/MIC 0010F
Feedlines and Tips, Pack of 50	I/ASS 8007F
Handpiece Handles, Pack of 5	I/ASS 8024F
Handpiece Tube Removal Tool	I/ASS 8625F
Handpiece - 0.4mm Green	I/ASS 8189F
Handpiece - 0.6mm Silver	I/ASS 8191F
Handpiece - 0.8mm Gold	I/ASS 8192F
500 ml Cutting Fluid Bottle (AquaSol)	I/ASS 8200F
500 ml Bottle (Empty)	I/ASS 8201F
Nozzle Cleaning Wire	I/ASS 8073F
Nozzle Wash Syringe, Pack of 2	I/ASS 8025F
Spare Parts	
Storage Tank and Lid	I/ASS 8193F
Dosing Chamber Lid	I/ASS 8003F
Powder Storage Pot	I/ASS 8193F
Twin Tube (with check valve)	I/ASS 8020F
Check Valve, Pack of 2	I/ASS 8004F
Foot Pedal Assembly	I/ASS 8153F
Dosing Chamber Starfish Seal	I/FIT 0012F
Complimentary Accessories	
Velopex Patient Treatment Centre	I/ASS 0045F
Velopex Dental Air Abrasion Cabinet	I/MAC 6200F

www.velopex.com